



CONTIPHARMA

a member of Bedelco group

PROTECT

Masques respiratoire FFP3 Purism



MARQUE

PURISM

EFFICACITÉ DE FILTRATION BACTÉRIENNE

≥ 95% (testé selon les normes GB2626-2019)

TAILLE

13,8x11x5x5,2 cm

SPÉCIFICATIONS

Masque 5 couches :

- tissu non tissé en PP
- tissu soufflé par fusion en PP
- tissu non tissé par air
- pince-nez en aluminium
- serre tête en nylon/polyuréthane composite



APPLICATION

Les respirateurs à particules Purism FFP3 offrent une protection respiratoire efficace pour une utilisation dans les industries où les travailleurs seront exposés à des particules solides (poussières) et / ou des particules liquides non volatiles. Idéal pour les applications où un facteur de protection plus élevé est requis, offrant une protection contre la plupart des gaz, vapeurs et particules.

CONFORMITÉ

Marquage CE 2016/425
Norme EN 149/2001+a1/2009

CONDITIONNEMENT

20 pces / boîte., 20 boîtes / carton, 400 pièces / carton,
Dimensions du carton : 67 × 28 × 36 cm – 11,7 kg

STOCKAGE

à stocker dans un endroit gazeux sec, propre, ventilé et non corrosif, loin de sources d'ignition ou de matériaux combustibles

DURÉE MAXIMUM DE STOCKAGE

3 ans



Wearing method

1. Hold the mask with your fingertips at the position of the nose clip to allow the headband to hang freely.
2. Put the mask around your chin with the nose clip facing upwards and wear the lower headband on the neck below the ears.
3. Put the upper headgear on the top of the head.
4. Use the index fingers of both hands to move from the middle of the nose clip to both sides and press down the nose clip until it is close to the bridge of the nose. One-handed operation is not recommended.
5. Before entering the work area, you must check the tightness of the mask and face, the inspection method is as follows:
 - a) Cover the mask with both hands, taking care not to affect the position of the mask on the face.
 - b) Take a deep breath. If you feel a leak near the bridge of the nose, adjust the nose clip again according to step 4;
 - c) If the leak is on the edge of the mask, the headgear should be readjusted.

Warning! If the mask and face cannot be in close contact, do not enter the work area. Please contact your supervisor.

Removal method

Referring to step 2 of the wearing method, hold the protective mask with one hand to maintain the wearing position, and use the other hand to pull down the headband and pull it over the top of the head. Continue to hold the mask with your hand, then pull the headband over the top of your head and remove the mask.

LOT
3 years

生产日期 2020/08/22
00098521

Shandong Daddy's Choice Health Science and Technology Co., Ltd
Address: Tiandingfeng road Garden street Economic Development Zone
Linyi County Dezhou City Shandong Province 251500 CN
Tel: 0534-8271567, 0534-8271568;
Website: www.daddychoice.com.cn
Email: sales@daddychoice.com
MADE IN CHINA

6 973601 650148

Restrictions on use

1. This mask does not generate oxygen and cannot be used in environments where the oxygen content is below 19.5%.
2. It cannot be used in an environment that momentarily endangers life and health.
3. It cannot be used in an environment where the pollution concentration or content exceeds the maximum concentration prescribed by the government or the allowable exposure concentration value is 10 times.
4. Cannot be used in environments where the concentration of oily particulate matter or organic vapor exceeds occupational limits.
5. Please do not change, wash, abuse or misuse this protective mask.
6. Do not use beards or other facial hair or conditions that impede a good seal between the face and the sealing surface of the protective mask.
7. This mask is designed for occupational protective equipment, and the user should be an adult who has received training in proper use methods and understands the restrictions on use. This mask is not designed for children.

Period of Validity:3 years
Production Date/Batch Number:Showing on the Packaging

Precautions

1. This product is only used for breathing protection against certain particulate matter. Misuse of protective masks will reduce its effectiveness and may cause disease or permanent damage to health or even death.
2. Before use, you should receive training on the correct method of use and follow the relevant safety and health standards.
3. Please carefully check whether all parts of the protective mask are intact before use. If damage occurs, the mask should be discarded or replaced.
4. This product is a disposable protective mask. When the mask is damaged during use or the breathing resistance is too high, be sure to replace the mask.
5. Use within the limits of applicable government regulations.
6. Leave the work area immediately if:
 - a) Feeling difficult to breathe;
 - b) Feeling sick and other discomforts.

[Storage conditions]

This product should be stored in a well-ventilated, dark and dry environment. Keep away from fire, pollutants and possible pollution.
Storage temperature range: -20°C to 38°C; storage humidity: <80% relative humidity. When transporting this product use original packaging provided.

-20°C 38°C
Temperature range of storage conditions Maximum relative humidity of storage conditions See information supplied by the manufacturer Do not re-use